


Ultrasonic activation

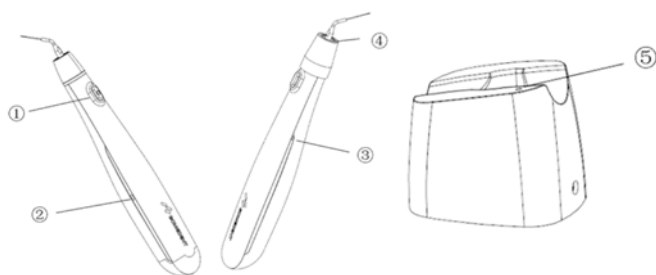

Product advantages

- 1、Two working model: Regular and high power options
- 2、Front Light : Led light gives a better view inside oral cavity
- 3、Wireless charging : Faster and more convenient
- 4、Bluetooth: Wireless foot control
- 5、45 kHz High Frequency
- 6、Battery 1600 MAH: Longer standby time
- 7、Autoclavable Tips, meet more root canal situation



Operating

Working Tip Installation

E98 Stainless Steel: #20/02 21mm 3pcs

EL96 Stainless Steel: #25/02 18mm 3pcs

Align the threads on the tip with the studs on the main unit, insert them together, make sure the threads are properly matched, and turn slightly clockwise.

Use the wrench to align the working tip slot, turn the wrench clockwise, and slowly apply force until the working tip is tightened.

After the working tip is tightened, it is normal to have a deviation within 10°.

Warning:

Turned off when installing the working tip;

Ensure working tip is securely installed, without loosening or falling off;

Please use the original working tip;

Working tip is not sterilized please use it after sterilization;

Working Tip Removing

Align the working tip slot with a wrench, turn the wrench counterclockwise to loosen the working tip, and continue to rotate until the working tip comes out.

Warning:

Working tip is used does not exceed 20 operations, and the number of 2 root canals in one operation is used as a reference.

When installing or removing the tip, be careful not to injure your fingers;

When installing or removing the tip, make sure that the main unit turned off.

Turn On/off

Press ① for 1 second, with a sounding, the device turn on/off

Operate:

After device turn on, press ① for 0.1 second, button turn to green, the device start working, press ① again, button off and device stop working

Power Change

Change working model from high to regular power: when the device is working, press ① for 1 second, then the button green light blinking all the time, means changed from high to regular power.

② Battery indicator light

Blue light: Battery power is above 50%;

Yellow light: Battery power is between 10%-50%;

Yellow light blinking : Battery power less than 10%, need charge

Green light: On charging, will auto turns off when fully charged.

③ Bluetooth function

Turn on: make sure the device is turn on, firstly press any button of foot control, then short press the Bluetooth button, will auto connect with foot control.

Turn off: short press the bluetooth button, will auto disconnect with foot control.

Bluetooth button Light indicator always on in blue: Bluetooth turned on, but not connected with foot control;

Bluetooth button Light indicator blue light blinking : connected with foot control and can work normally;

Bluetooth button Light indicator off: Bluetooth function is not turned on.

④ Lighting

During working motion, lighting always on.

⑤ Charging base indicator light

Blue means working on.

Keep it for 30 to 60 seconds each time to get the best cleaning effect.